Dainamed®

Scar Sun Protector

THIN SILICONE SHEET
PLANCHA DE SILICONA FINA

 **UPF 50+**

Instructions for use Instrucciones de uso

EN Silicone sheet to improve the appearance of skin with visible marks / **ES** Lámina de silicona para mejorar el aspecto de la piel con marcas visibles / **FR** Feuille de gel de silicone pour les soins des cicatrices / **DE** Silikonfolie zur Verbesserung des Hautbildes bei sichtbaren Hautveränderungen / **IT** Foglio in silicone per migliorare l'aspetto della pelle con segni visibili / **PT** Folha de silicone para melhorar o aspeto da pele com marcas visíveis / **RU** Силиконовая накладка для улучшения внешнего вида кожи с заметными следами / **SV** Silikonark för att förbättra utseendet på hud med synliga märken / **FI** Silikonilevy, joka parantaa ihon ulkonäköä, kun siinä on näkyviä jälkiä / **AR** لاصقة سيليكون جل للندبات / **VN** Miếng silicon giúp cải thiện vẻ ngoài của làn da có các vết sẹo rõ rệt / **עברית** פלטת סיליקון לטיפול בצלקות / **CZ** Silikonová fólie pro zlepšení vzhledu pokožky s viditelnými stopami / **SK** Silikonová fólia na zlepšenie vzhľadu pokožky s viditeľnými stopami

Reference	Size	Units	Shape	Type
F-02565-51CDA	5 x 7,5 cm	5		Skin feel
F-02565-54CDA	4 x 30 cm	5		Skin feel
F-02565-55CDA	10 x 18 cm	5		Skin feel

				
EN	Measure	Cut	Remove plastic	Apply
ES	Medir	Cortar	Retirar plástico	Aplicar
FR	Mesure	Coupez	Retirez le plastique	Appliquez
PT	Medir	Cortar	Retirar o plástico	Aplicar
IT	Misurare	Tagliare	Rimuovere la plastica	Applicare
DE	Mät	Klipp	Ta bort plast	Applicera
AR	القياس	قص	إزالة البلاستيك	تطبيق
RU	Измерьте	Отрежьте	Снимите пластиковую пленку	Наложите
SV	Mät	Beskär	Ta bort plastfilmen	Applicera
FI	Mittaa	Leikkaa	Poista muovikelmu	Kiinnitä
VN	Đo lường	Cắt	Loại bỏ nhựa	Áp dụng
CZ	Poměřte	Zastříhněte	Odstraňte fólii	Aplikujte
SK	Zmerajte	Upravte	Odstráňte fóliu	Aplikujte

Click here for **English**
 Clic aquí para **Español**
 Cliquez ici pour **Français**
 Clique aqui para **Português**
 Clicca qui per l'**italiano**
 Klicken Sie hier für **Deutsch**
 انقر هنا للغة العربية
 Нажмите здесь для **русского** языка
 Klicka här för **Svenska**
 Klikkaa tästä **suomen kieleen**
 Bấm vào đây để đọc tiếng **Việt**
 תירבעל ואב ץחל
 Klikněte pro **Češtinu**
 Kliknite pre **Slovenčinu**

Thin Silicone sheets

SKIN FEEL
Indicated to improve the appearance of skin with visible marks, both recent and old, regardless of their origin.
UPF 50 sun protection
0,4mm thickness (+/-1%)
Semi-transparent beige: skin-like feel
Lighter and thinner, the best choice for everyday wear

1. Description

These adhesive silicone sheets are designed to help improve the appearance of skin with visible marks, helping to keep it hydrated and protected daily, including protection against solar radiation (UPF 50).

- They help the skin look smoother and more even.
- They promote skin hydration.
- Washable and reusable up to 20 times, depending on care.
- The sheet provides a physical barrier with UV protection (UPF 50).

2. Who is it for?

Suitable for individuals with visible marks on the skin who wish to improve their appearance.

3. How does it work?

- The sheet adapts to the skin, forming a protective layer that helps maintain hydration.
- It contributes to improving skin comfort in the application area.
- Its flexible, elastic, and soft material allows for comfortable application, even on areas subject to movement.

4. Instructions for use

1. Apply to clean, dry skin.

2. Remove the transparent protective film before use.

3. Place the sheet over the desired area, ensuring proper adhesion.

4. If necessary, trim to adjust the size before application.

5. Cleaning and reuse

1. Clean the sheet with lukewarm water and mild soap after each use.
2. Before reusing, let the adhesive side air dry to facilitate drying.
3. Replace the product when it loses its stickiness or shows signs of wear.

6. Warnings and precautions

- For external use only.
- Do not apply to irritated, damaged, or broken skin (open wounds).
- In case of irritation, discontinue use.
- Keep out of reach of children. Do not swallow.
- Clean the sheet properly before reusing it.

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**FRESCO INTERNATIONAL 2005, S.A.**